

BC BIOMEDICAL CS-2000 SERIES TABLE OF CONTENTS

WARNINGS, CAUTIONS, NOTICES	ii
DESCRIPTION	1
LAYOUT	2
OPERATION	4
TESTING	6
MANUAL REVISIONS	7
LIMITED WARRANTY	7
SPECIFICATIONS	8
NOTES	9

This User Manual covers the following units:

- CS-2000-U
- CS-2000-E

WARNING - USERS

The CS-2000 is for use by skilled technical personnel only.

WARNING - USE

The CS-2000 is intended for testing only and should never be used in diagnostics, treatment or any other capacity where they would come in contact with a patient.

WARNING - CONNECTIONS

All connections to patients must be removed before connecting the DUT to the CS-2000. A serious hazard may occur if the patient is connected when testing with the unit. Do not connect any leads from the patient directly to the unit or DUT while it is powered by the Analyzer.

WARNING - CLEANING

Disconnect Line Power to the CS-2000 before attempting to clean it. Do not immerse. The unit should be cleaned by wiping gently with a damp, lint-free cloth. A mild detergent can be used if desired.

WARNING - LIQUIDS

Do not submerge or spill liquids on the CS-2000. Do not operate the CS-2000 if internal components not intended for use with fluids may have been exposed to fluid, as the internal leakage may have caused corrosion and be a potential hazard.

WARNING - MODIFICATIONS

The CS-2000 is intended for use within the published specifications. Any application beyond these specifications or any unauthorized user modifications may result in hazards or improper operation.

CAUTION - USAGE

The CS-2000 is not a continuous duty device, it is intended for short duration testing within the current limits specified. Do not leave the DUT connected to this unit for extended time periods.

CAUTION - ENVIRONMENT

Exposure to environmental conditions outside the specifications can adversely affect the performance of the CS-2000. If the unit is outside of Operating specifications, allow it to acclimate to specified conditions for at least 30 minutes before attempting to operate it.

CAUTION - INSPECTION

The CS-2000 should be inspected before each use for obvious signs of abuse or wear. The unit should not be used and should be serviced if any parts are in question.

CAUTION - SERVICE

The CS-2000 is intended to be serviced only by authorized service personnel. Troubleshooting and service procedures should only be performed by qualified technical personnel.

NOTICE – SYMBOLS

Symbol Description



20001191101

Caution (Consult Manual for Further Information)

Electrical Caution

(Consult Manual for further information)

NOTICE – ABBREVIATIONS

A or Amp	Ampere(s)
AAMI	Association for the Advancement of Medical Instrumentation
С	Celsius
0	Degree(s)
DUT	Device Under Test
Euro	European
Hz	Hertz
IEC	International Electrotechnical Commission
kg	kilogram(s)
mm	millimeter(s)
Ω	ohm(s)
lbs	pounds
RH	Relative Humidity
RMS	Root Mean Square
S	second(s)
USA	United States of America
v	Volt(s)
VA	Volt-Ampere(s)
VAC	Volt(s) Alternating Current
w	Watt(s)

NOTICE – DISCLAIMER

BC GROUP INTERNATIONAL, INC. WILL NOT BE RESPONSIBLE FOR ANY INJURIES SUSTAINED DUE TO UNAUTHORIZED EQUIPMENT MODIFICATIONS OR APPLICATION OF EQUIPMENT OUTSIDE OF THE PUBLISHED INTENDED USE AND SPECIFICATIONS.

NOTICE – DISCLAIMER

BC GROUP INTERNATIONAL, INC. RESERVES THE RIGHT TO MAKE CHANGES TO ITS PRODUCTS OR SPECIFICATIONS AT ANY TIME, WITHOUT NOTICE, IN ORDER TO IMPROVE THE DESIGN OR PERFORMANCE AND TO SUPPLY THE BEST POSSIBLE PRODUCT. THE INFORMATION IN THIS MANUAL HAS BEEN CAREFULLY CHECKED AND IS BELIEVED TO BE ACCURATE. HOWEVER, NO RESPONSIBILITY IS ASSUMED FOR INACCURACIES.

NOTICE – CONTACT INFORMATION

BC BIOMEDICAL BC GROUP INTERNATIONAL, INC. 3081 ELM POINT INDUSTRIAL DRIVE ST. CHARLES, MO 63301 USA

> 1-800-242-8428 1-314-638-3800

www.bcgroupintl.com sales@bcgroupintl.com

CS-2000 Series User Manual www.bcgroupintl.com 06/12 Copyright © 2012 Made in the USA Rev 05 This Page Intentionally Left Blank

BC BIOMEDICAL CS-2000 SERIES 1 AMP CURRENT SOURCE

The CS-2000 Series is a 1 Amp Current Source for testing the ground continuity in equipment. This device facilitates testing to IEC601 requirements, where a 1 A source current is required. This device interfaces directly with BC Biomedical SA-2000 series of Safety Analyzers as well as the Dale 544D & 544D Lite, and Dale 601 & 601E Safety Analyzers.

CS-2000 SERIES:

- FACILITATES TESTING TO IEC601 REQUIREMENTS
- INTERFACES DIRECTLY TO BC BIOMEDICAL SA-2000 SERIES OF SAFETY ANALYZERS
- INTERFACES TO DALE 544D, 544D LITE, 601 AND 601E SAFETY ANALYZERS
- 90 264 VAC OPERATION (50/60 Hz)
- 1 AMP CURRENT SOURCE FOR RESISTANCE MEASUREMENTS
- NO PROTRUDING PUSH BUTTONS
- LIGHT TOUCH KEY USED TO OPERATE
- SMALL SIZE

AVAILABLE MODELS:

- CS-2000-U: STANDARD MODEL WITH HOSPITAL-GRADE NEMA 5-15P LINE PLUG AND HOSPITAL-GRADE NEMA 5-15R DUT TEST RECEPTACLE FOR USE IN THE USA AND OTHER COMPATIBLE COUNTRIES.
- CS-2000-E: EUROPEAN MODEL WITH SCHUKO LINE PLUG AND SCHUKO DUT RECEPTACLE FOR USE IN APPLICABLE EUROPEAN COUNTRIES.

ACCESSORIES

- BC20 20221 SCHUKO GROUNDING ADAPTER
- BC20 20114 REPLACEMENT KELVIN LEAD FOR CS-2000
- BC20 30106 SOFT-SIDED CARRYING CASE

LAYOUT

This section looks at the layout of the CS-2000 and gives descriptions of the elements that are present.



CS-2000-U Layout



CS-2000-E Layout

OPERATION

The CS-2000 1 Amp current source is not a stand-alone device. It is designed to work in conjunction with a BC Biomedical SA-2000 Series Safety Analyzer or a DALE 544D, 544D Lite, 601, and 601E Safety Analyzer.

Push To Test Key

Pressing and holding this key initiates a test.

This applies the 1 Amp current source to the Device Under Test (DUT).

LED Indicator

This LED gives the operator visual feedback on the test status.

The following table explains the meaning of each possible LED status indication:

LED Status	Meaning		
OFF	Test is not functional		
ON	Test is Valid		
BLINKING	Resistance is greater than 3 ohms		

Connections

- 1. The Line plug on the CS-2000 plugs into the Safety Analyzer DUT Receptacle.
- 2. The Chassis cable, located at the top-right corner of the CS-2000, plugs into the Chassis Connection port on the Safety Analyzer.
- 3. The DUT plugs into the CS-2000 Receptacle cable.
- 4. The detachable cable, located at the bottom-right corner of the CS-2000, is used to connect to the chassis of the DUT.

Below is a typical example for wiring configuration of the Safety Analyzer (BC International SA-2005 shown), the CS-2000 (CS-2000-U model shown) Current Source and the DUT (in this case a Patient Monitor).



TESTING

WARNING

All connections to patients must be removed before connecting the DUT to the CS-2000 or Analyzer. A serious hazard may occur if the patient is connected when testing with the Analyzer. Do not connect any leads from the patient directly to the CS-2000 or Analyzer from the DUT.

- Connect the CS-2000 to the Safety Analyzer and the DUT as indicated in the Connections portion of the Overview Section.
- 2. Set the Safety Analyzer for Forward Polarity with Hot and Neutral Closed.
- Set the Safety Analyzer to read earth resistance. The safety analyzer display will read approximately 0.67 ohms.

NOTE: When the CS-2000 is connected to the Safety Analyzer and the Push to Test button is not held, bias voltages cause a reading to be displayed on the Safety Analyzer. This reading is approximately 0.67 ohms. This is not a true reading of the DUT. Valid tests are indicated only when the CS-2000 Status LED is continuously lit; readings on the Safety Analyzer in all other situations should be ignored.

- 4. On the CS-2000, press and hold the Push to Test key.
- 5. Verify the validity of the test by checking the LED on the CS-2000.

NOTE: A continuously lit LED indicates a valid test.

6. Read the resistance on the Safety Analyzer.

MANUAL REVISIONS

Revision # Revisions Made

- Rev 01 Origination
- Rev 02 Updated Pictures
- Rev 03 Specifications Updated, Misc. Edits
- Rev 04 Format Updated, Pictures Updated, Misc. Edits
- Rev 05 Misc. Edits

LIMITED WARRANTY

WARRANTY: BC GROUP INTERNATIONAL, INC. WARRANTS ITS NEW PRODUCTS TO BE FREE FROM DEFECTS IN MATERIALS AND WORKMANSHIP UNDER THE SERVICE FOR WHICH THEY ARE INTENDED. THIS WARRANTY IS EFFECTIVE FOR TWELVE MONTHS FROM THE DATE OF SHIPMENT.

EXCLUSIONS: THIS WARRANTY IS **IN LIEU OF** ANY OTHER WARRANTY EXPRESSED OR IMPLIED, INCLUDING, BUT NOT LIMITED TO ANY IMPLIED WARRANTY OF **MERCHANTABILITY** OR FITNESS FOR A PARTICULAR PURPOSE.

BC GROUP INTERNATIONAL, INC. IS NOT LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES.

NO PERSON OTHER THAN AN OFFICER IS AUTHORIZED TO GIVE ANY OTHER WARRANTY OR ASSUME ANY LIABILITY.

REMEDIES: THE PURCHASER'S SOLE AND EXCLUSIVE REMEDY SHALL BE: (1) THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS OR PRODUCTS, WITHOUT CHARGE. (2) AT THE OPTION OF **BC GROUP INTERNATIONAL, INC.**, THE REFUND OF THE PURCHASE PRICE.

P:\MANUALS\BCGroup\...\CS2000_UM_Rev05.doc

SPECIFICATIONS

Electrical						
CURRENT SOURCE		1.00 to 1.02 A DC				
OPERATING LINE VOLTAGE		90 to 264 VAC, 50/60 Hz				
POWER CONSUMPTION		12 VA (12 W)				
LOAD CURRENT		This device is an accessory to a Safety Analyzer, refer to applicable Safety Analyzer User Manual for this Rating				
LINE PLUG	CS-2000-U	NEMA 5-15P				
	CS-2000-E	European CEE 7/7 "Schuko"				
DUT	CS-2000-U	Hospital Grade NEMA 5-15R				
RECEPTACLE	CS-2000-E	European CEE 7/7 "Schuko"				
CONNECTIONS		3 Pin Locking Plug for Safety Analyzer Chassis Cable Connection (See Description section for Compatible Safety Analyzers) Detachable Kelvin Cable (BC20-20114) for				
		DUT Ground Connection				

Physical					
CONSTRUCTION	ENCLOSURE	ABS Plastic			
CONSTRUCTION	OVERLAY	Back-printed Lexan			
SIZE	7.38 x 3.94 x 1.64 Inches (187.5 x 100.1 x 41.7 mm)				
WEIGHT	< 1 Lbs (0.45 kg)				
OPERATING RANGE	0 to 50 °C (32 to 122 °F) 10 to 90% RH, Non-Condensing				
STORAGE RANGE -40 to 60 °C (-40 to 140 °F)		-40 to 140 °F)			

NOTES

NOTES



BC GROUP INTERNATIONAL, INC. 3081 ELM POINT INDUSTRIAL DRIVE ST. CHARLES, MO 63301 USA

> 1-800-242-8428 1-314-638-3800

www.bcgroupintl.com sales@bcgroupintl.com

CS-2000 Series User Manual 06/12 – Rev 05

> Copyright © 2012 Made in the USA