




ForceTriad™ PM Procedure Using ESU-2400 Autosequence

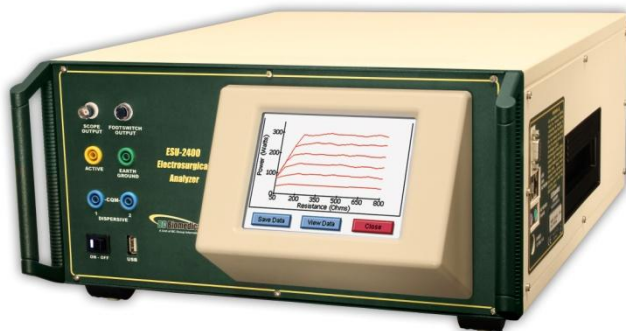

ESU-2400



ForceTriad™

USER GUIDE

**BC BIOMEDICAL
FORCETRIAD™ PM AUTOSEQUENCE
TABLE OF CONTENTS**

| | |
|---|----|
| WARNINGS, CAUTIONS, NOTICES | ii |
| OVERVIEW | 1 |
| EQUIPMENT LIST | 2 |
| LOADING THE AUTOSEQUENCE | 3 |
| RUNNING THE AUTOSEQUENCE | 6 |
| PHYSICAL INSPECTION..... | 8 |
| LOW VOLTAGE SUPPLY TEST | 9 |
| REM TEST | 10 |
| CROSS COUPLING TESTS | 11 |
| MONOPOLAR POWER OUTPUT TESTS | 15 |
| AUTOBIPOLAR AND BIPOLAR OUTPUT TESTS..... | 18 |
| LIGASURE™ POWER OUTPUT TESTS..... | 19 |
| HIGH FREQUENCY LEAKAGE TESTS..... | 20 |
| LOW FREQUENCY LEAKAGE TESTS | 25 |
| VALLEYLAB EXCHANGE | 25 |
| USER GUIDE REVISIONS | 26 |
| NOTES..... | 27 |

WARNING - USERS

The ESU-2400 is for use by skilled technical personnel only.

WARNING - USE

The ESU-2400 is intended for testing only and should never be used in diagnostics, treatment or any other capacity where it would come in contact with a patient.

WARNING - USE

Never touch exposed metal surfaces on test leads or other current-carrying parts while the ForceTriad™ is activated.

WARNING - MODIFICATIONS

The ESU-2400 is intended for use within the published specifications. Any application beyond these specifications or any unauthorized user modifications may result in hazards or improper operation.

WARNING - CONNECTIONS

All connections to patients must be removed before connecting the ForceTriad™ to the ESU-2400. A serious hazard may occur if the patient is connected when testing with the ESU-2400. Do not connect any leads from the patient directly to the ESU-2400 or ForceTriad™.

NOTICE – TRADEMARKS

Valleylab™, ForceTriad™, LigaSure™, REM™, and ForceTriVerse™ are trademarks of Covidien.

Copyright © 2012 Covidien. All rights reserved. Reprinted with the permission of the Surgical Solutions business unit of Covidien.

NOTICE – LICENSE AGREEMENT

The software used to perform the ForceTriad remote control is licensed by BC Biomedical from Energy-based Devices, a global business unit of Tyco Healthcare Group LP, a Delaware Limited partnership d/b/a Covidien on behalf of itself and certain of its affiliates. The end user is permitted to use the software solely for the purpose of performing the PM procedure. The end user shall not use the Autosequence in any clinical setting or in the provision of medical care to, or the treatment of, any party in any way whatsoever.

Covidien reserves all rights in and to the Covidien Platform and Covidien Software and any modifications thereto and derivations thereof, including, but not limited to, all title, ownership, intellectual property rights and all other rights and interests.

NOTICE – DISCLAIMER

BC GROUP INTERNATIONAL, INC. WILL NOT BE RESPONSIBLE FOR ANY INJURIES SUSTAINED DUE TO UNAUTHORIZED EQUIPMENT MODIFICATIONS OR APPLICATION OF EQUIPMENT OUTSIDE OF THE PUBLISHED INTENDED USE AND SPECIFICATIONS.

NOTICE – DISCLAIMER

BC GROUP INTERNATIONAL, INC. RESERVES THE RIGHT TO MAKE CHANGES TO ITS PRODUCTS OR SPECIFICATIONS AT ANY TIME, WITHOUT NOTICE, IN ORDER TO IMPROVE THE DESIGN OR PERFORMANCE AND TO SUPPLY THE BEST POSSIBLE PRODUCT. THE INFORMATION IN THIS MANUAL HAS BEEN CAREFULLY CHECKED AND IS BELIEVED TO BE ACCURATE. HOWEVER, NO RESPONSIBILITY IS ASSUMED FOR INACCURACIES.

NOTICE – CONTACT INFORMATION

BC BIOMEDICAL
BC GROUP INTERNATIONAL, INC.
3081 ELM POINT INDUSTRIAL DRIVE
ST. CHARLES, MO 63301
USA


1-800-242-8428
1-314-638-3800

www.bcgrouintl.com
sales@bcgrouintl.com


OVERVIEW

This guide walks the operator through the Autosequence for the Covidien ForceTriad™ PM procedure. The cables and accessories used in this Autosequence come from the standard ESU-2400 accessory kit (BC20-00130) and the optional ForceTriad™ PM Accessory kit (BC20-00131).

Standard Accessory Kit, BC20-00130:

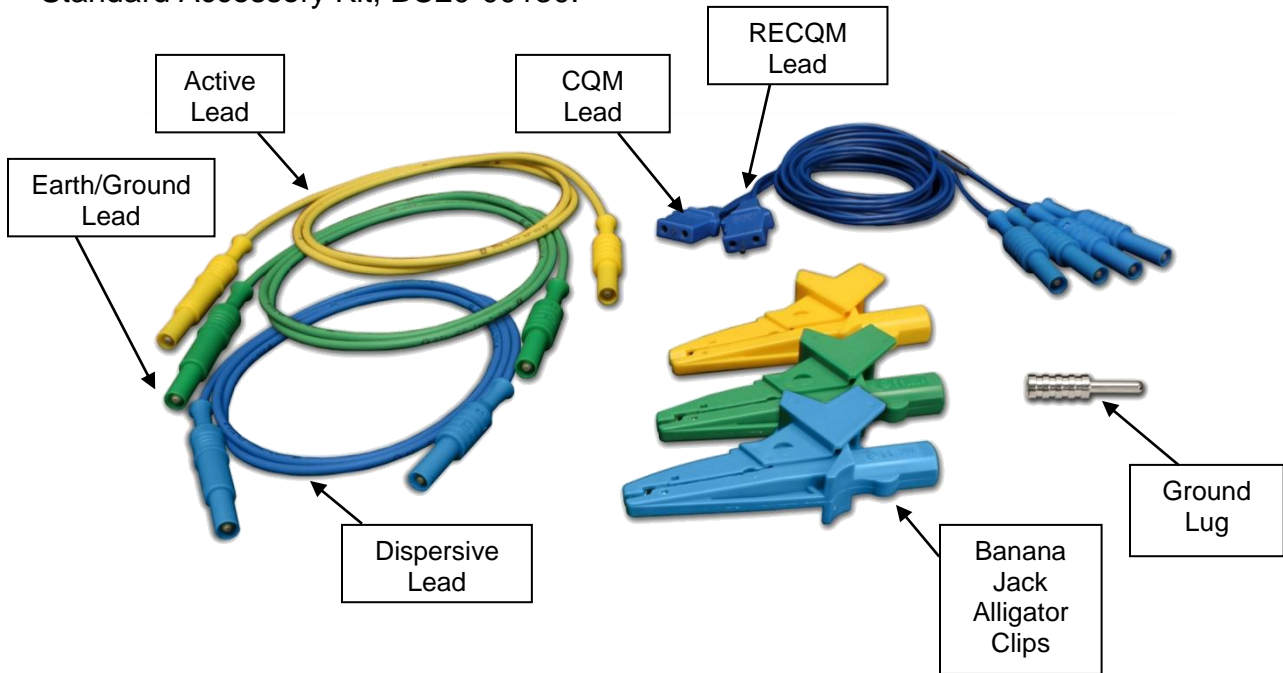

Optional ForceTriad™ PM Accessory Kit, BC20-00131

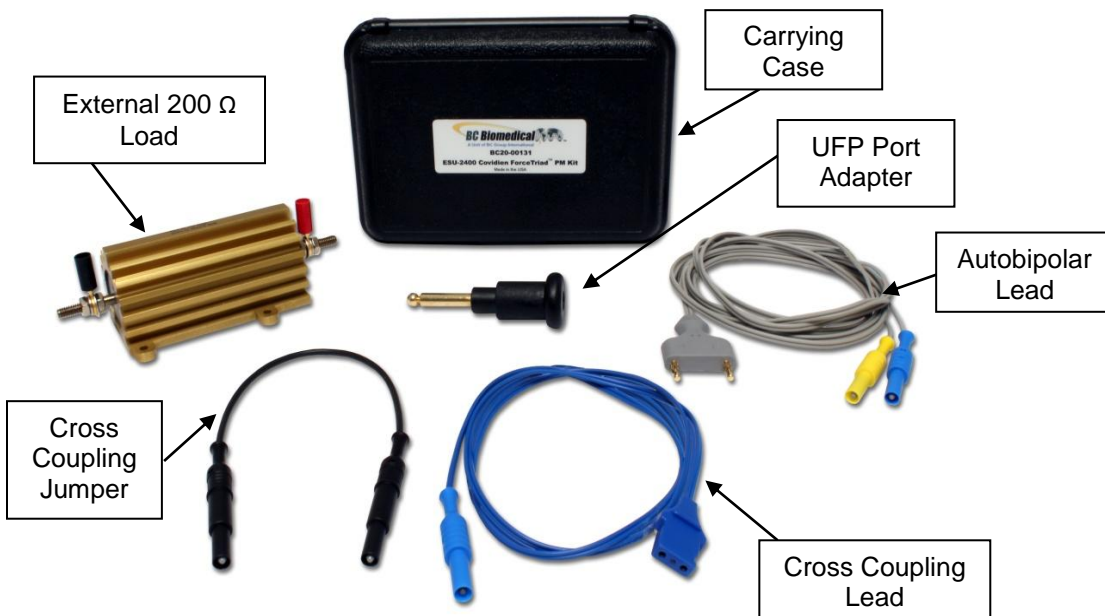


EQUIPMENT LIST

The following equipment is required to perform the ForceTriad™ PM procedure using the ESU-2400:


- Covidien ForceTriad™ (DUT, Software Version 3.50)
- BC Biomedical ESU-2400 (Software Version 1.0.3.15 or higher)
- ForceTriad™ Monopolar Footswitch
- ForceTriad™ Bipolar Footswitch
- ForceTriad™ LigaSure™ Handpiece
- ForceTriad™ LigaSure™ Footswitch
- Standard two-button or ForceTriVerse™ electrosurgical pencil
- UFP Adapter (included in BC20-00131 accessory kit)
- External 200 Ohm 250 Watt Load (included in BC20-00131 accessory kit)
- RS-232 Serial Communications cable (comes with ESU-2400)
- Test cables from BC20-00130 accessory kit (comes with ESU-2400)
- Test cables from BC20-00131 accessory kit (recommended)
- Digital Multi Meter (DMM)
- Safety Analyzer (for low frequency leakage testing) (BC recommends the BC Biomedical SA-2005)

LOADING THE AUTOSEQUENCE


Starting from the ESU-2400 Main Screen, press Autosequences

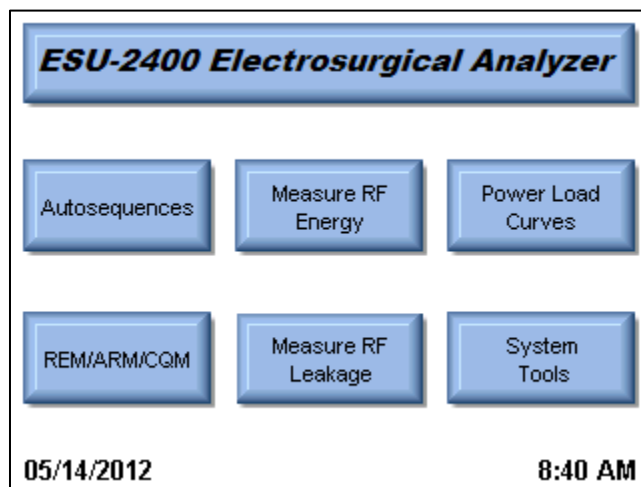


From the autosequence menu, press Select Autosequence

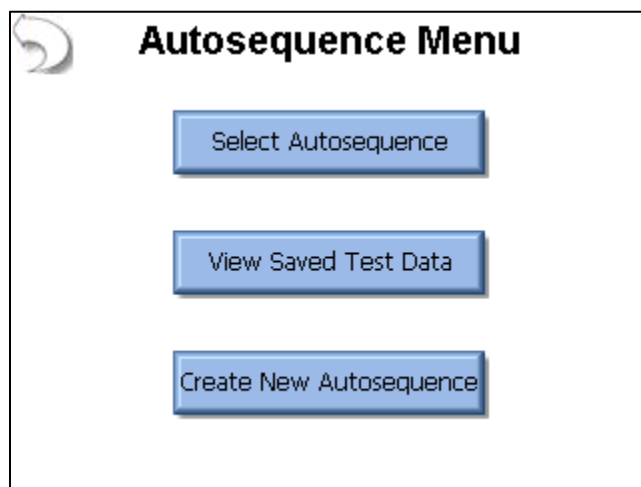

The open file filter will show Autosequence Setup (*.seq) by default. Change the filter to show Secure Sequence (*.ssq).

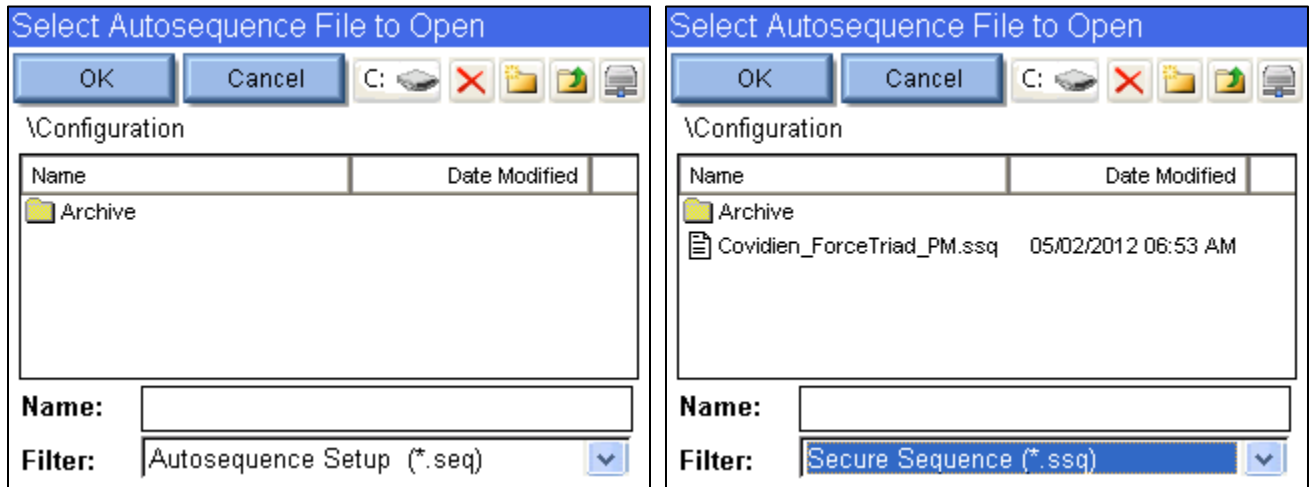

Select the file Covidien_ForceTriad_PM.ssq and press OK. After the sequence is loaded, the screen will display the list of tests that will be performed. Press the Start Test button to begin the PM procedure.

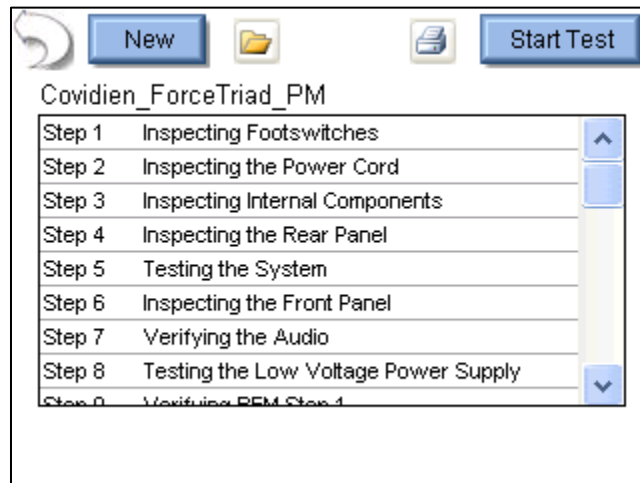

You will be prompted to enter information about the ForceTriad™. You can enter information by pressing on the text entry box. You can use the on-screen keyboard, USB keyboard, or USB barcode reader to enter information about the ForceTriad™. Press the Right Arrow to begin the test.

**Please Enter the following information
for the test report.**

Generator ID:


Manufacturer:

Model:

Serial Number:



Tested By:







Signature:
(Tap to insert
signature)



Autosequence Step 1/102 Operator Instruction


Inspecting Footswitches

1. Remove the footswitch from the system. 
2. Inspect the connector for damage or corrosion.
3. Inspect the footswitch for damage.
4. Reconnect the footswitch. 

   Show Meter  Pass 
 Show Text Fail


RUNNING THE AUTOSEQUENCE

When running the Autosequence, the screen of the ESU-2400 has two display modes. One is for showing the instructions to the user and the other is for showing the measurement. The following is an example of these screens along with a description of each part.

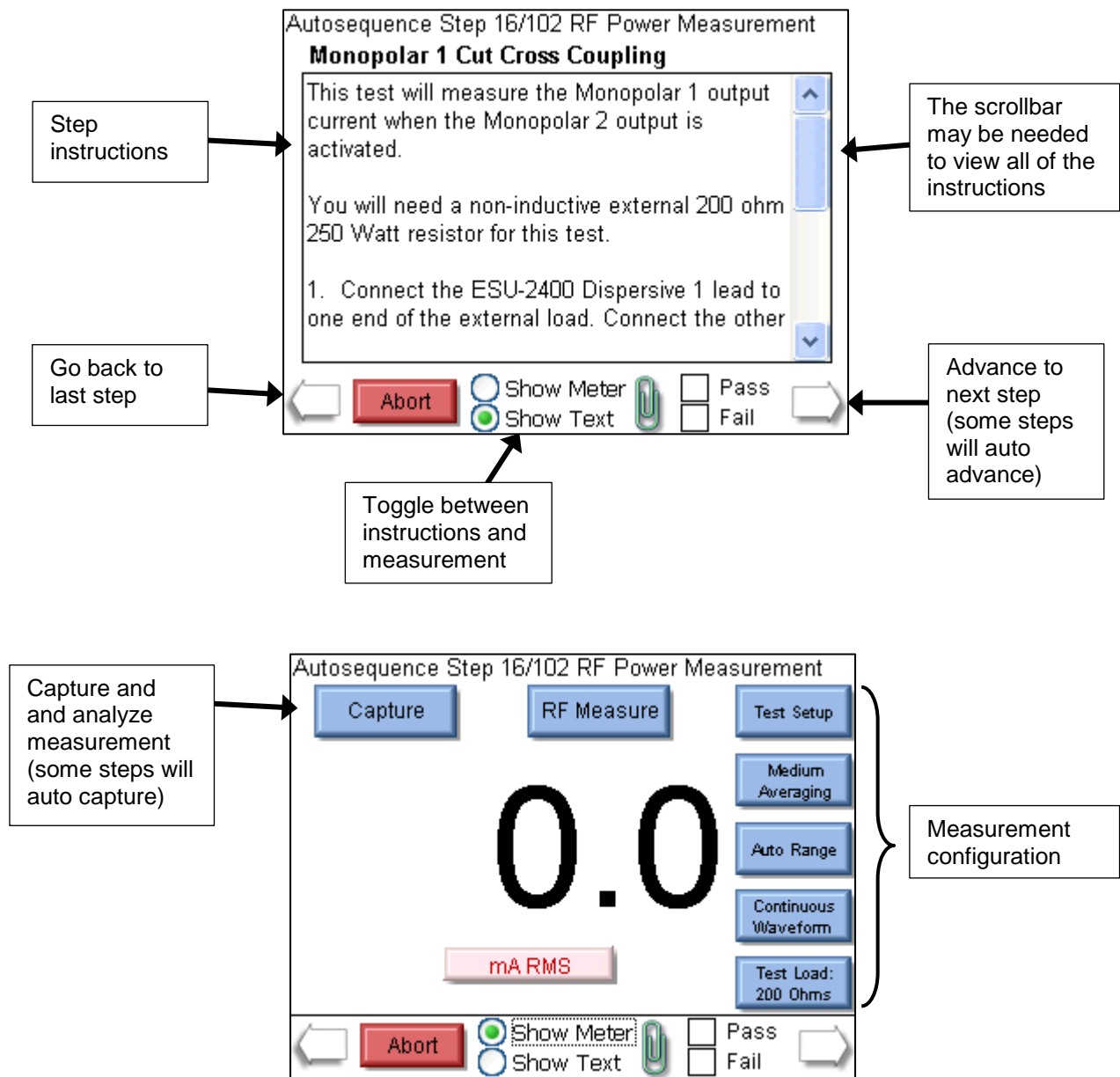

In this section of the guide each step of the Autosequence is separated into three blocks; the Step Header, Instructions, and Connections. The step header identifies the basic function of the step and which step or steps are being described.

| Step Number(s) | Step Description |
|---------------------|---|
| Instructions | These instructions tell the user what needs to be done to perform the step. These instructions are a summary of what is shown on the ESU-2400 screen. The user should follow the instructions on the screen and use the guide as support for how to interface with the ForceTriad™. |
| Connections | This section shows the user how to make the connections between the ESU-2400 and ForceTriad™. Not every step description has this section |

| | |
|---|-------------------------|
| Step Number 1-2 | ForceTriad™ Inspections |
| <p>Instructions These steps involve physical inspection of the footswitches and power cord. Follow the ESU-2400 on-screen instructions to perform these steps.</p> <p>On each step, use the scroll bar on the right side of the screen to view and follow the instructions. After you have completed the instructions, press on the Pass or Fail checkbox to mark the results of the test. Then press the Right Arrow to proceed to the next step.</p> | |
| Step Number 3 | Internal Inspections |
| <p>Instructions These steps involve physical inspection of the internal components and connectors of the generator and generator wiring. Follow the ESU-2400 on-screen instructions to perform this step.</p> <p>Use the scroll bar on the right side of the screen to view and follow the instructions. After you have completed the instructions, press on the Pass or Fail checkbox to mark the results of the test. Then press the Right Arrow to proceed to the next step</p> | |
| <p>Connections You will need to remove the generator's cover for this step. There are four screws on the back of the generator that need to be removed before the cover can be slid back and off the generator.</p> | |
|  | |
| Step Number 4-7 | ForceTriad™ Inspections |
| <p>Instructions These steps involve physical inspection of the generator and generator accessories. You will be checking the front panel connectors, rear panel connectors, footswitches, accessories, and internal components of the generator. Follow the on-screen instructions to perform these steps.</p> <p>On each step, use the scroll bar on the right side of the screen to view and follow the instructions. After you have completed the instructions, press on the Pass or Fail checkbox to mark the results of the test. Then press the Right Arrow to proceed to the next step</p> | |

Step Number 8

Testing Low Voltage Power Supply

Instructions


You will need a DMM to measure internal ForceTriad™ power supplies. The connections are described on the screen. The following pictures may help in finding the required test points.

Connections

+5V Supply



+12V Supply

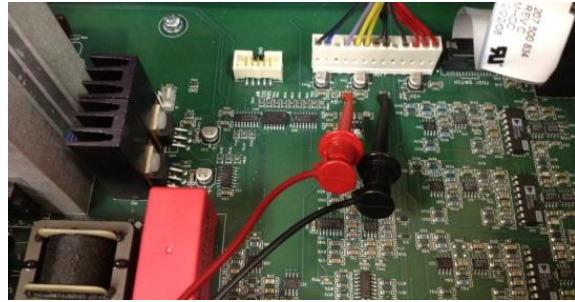

-12V Supply



After the supplies have been measured, Turn Off the ForceTriad™, reinstall the ForceTriad™ Cover and Connect the Serial cable between the ESU-2400 and ForceTriad™.




Turn the ForceTriad™ back on and wait for the system to finish booting. Then select the Pass or Fail status of the step and the right arrow to advance to the next step.

Step Number 9

Verifying REM

Instructions

Wait for the ESU-2400 to configure the ForceTriad™.

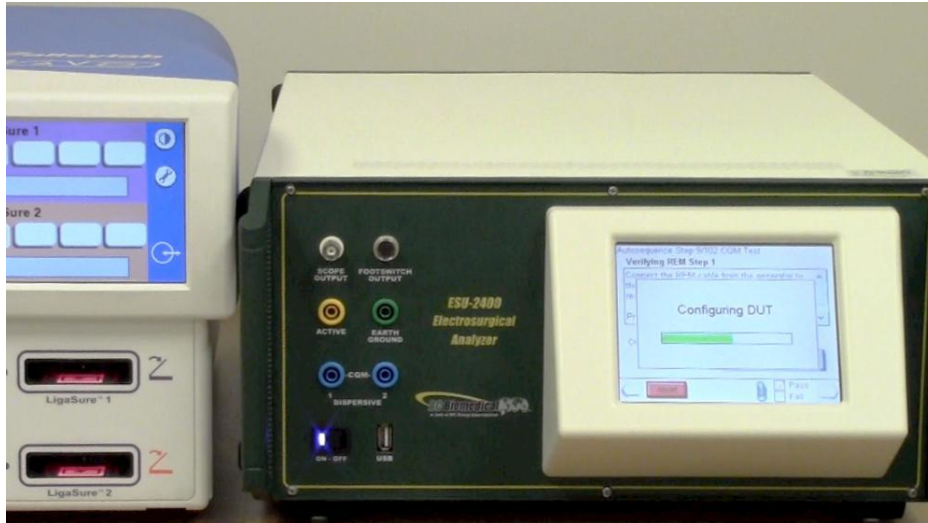

Connect the REM cable from the ForceTriad™ to the ESU-2400 Dispersive 1 and the 2 ports. On the ESU-2400 press Run Auto CQM Test.

Connections



Step Number 10-15

Verifying REM

Instructions

No action from user is required


Step Number 16

Monopolar 1 Cut Cross Coupling

Instructions

Connect the Monopolar 1 output from ForceTriad™ to one end of the 200 ohm external load. Connect the other end of the external load to the ESU-2400 Dispersive 1 port. Connect the ForceTriad™ REM port to the ESU-2400 Active port. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

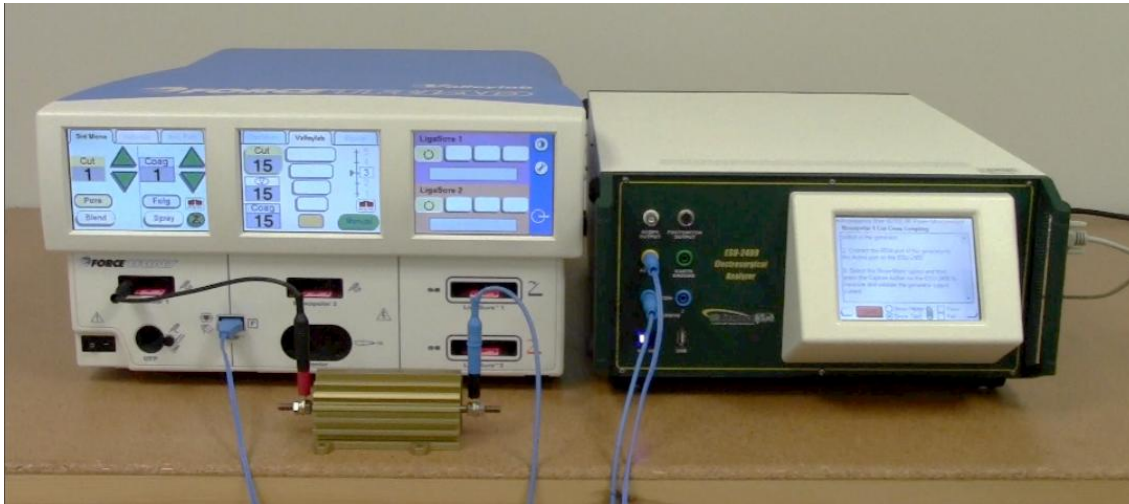

Step Number 17

Monopolar 1 Spray Cross Coupling

Instructions

No action from user is required.

Step Number 18

Monopolar 2 Cut Cross Coupling

Instructions

Move the short jumper lead from Monopolar 1 output to Monopolar 2 output. Don't change any connections on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



Step Number 19

Monopolar 2 Spray Cross Coupling

Instructions

No action from user is required.

Step Number 20


UFP Cut Cross Coupling

Instructions

Move the short jumper lead from Monopolar 2 output to the UFP output port using the UFP Adapter in the BC20-00131 accessory kit. Don't change any connections on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



| | |
|--|------------------------------|
| Step Number 21 | UFP Spray Cross Coupling |
| <p>Instructions</p> <p>No action from user is required.</p> | |
| Step Number 22 | Bipolar Cut Cross Coupling |
| <p>Instructions</p> <p>Remove the UFP adapter from the UFP port. Connect the ESU-2400 Active lead to one end of the external load. Connect the other end of the external load to the left tine Bipolar port. Connect the right tine Bipolar output of the ForceTriad™ to the Dispersive 1 port on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 23 | Bipolar Spray Cross Coupling |
| <p>Instructions</p> <p>No action from user is required.</p> | |


Step Number 24

Ligasure™ 1 Cross Coupling

Instructions

Move the short jumper lead from left tine Bipolar port to the left tine LigaSure 1 output of the ForceTriad™. Connect the right tine LigaSure 1 output of the ForceTriad™ to the Dispersive 1 port on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

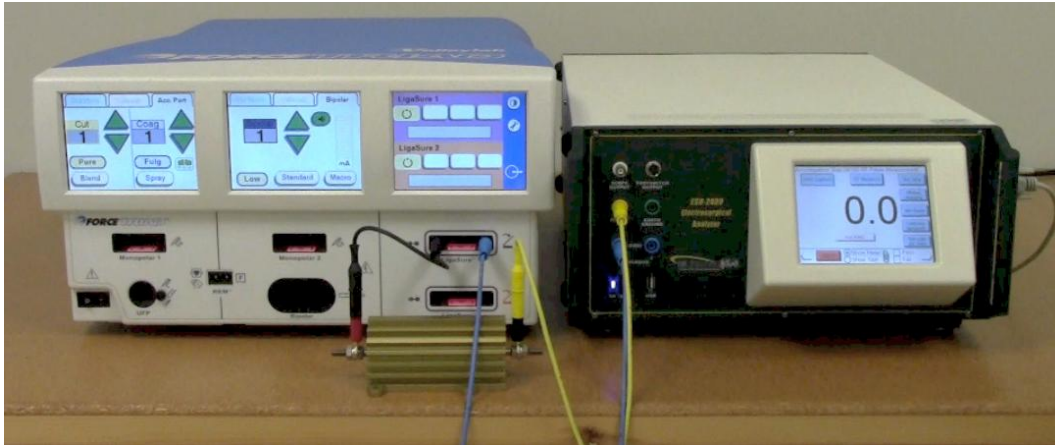

Step Number 25

LigaSure™ 1 Spray Cross Coupling

Instructions

No action from user is required.


Step Number 26

LigaSure™ 2 Cross Coupling

Instructions

Move the short jumper from the LigaSure 1 output of the ForceTriad™ to the left tine LigaSure 2 output of the ForceTriad™. Connect the right tine LigaSure 2 output of the ForceTriad™ to the Dispersive 1 port on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

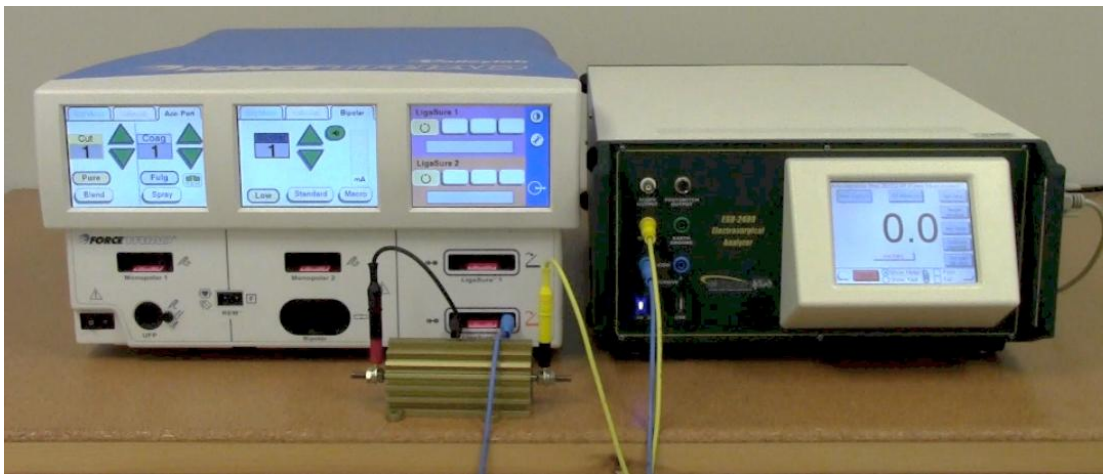

Step Number 27 LigaSure™ 2 Spray Cross Coupling

Instructions
No action from user is required.

Step Number 28 Monopolar 1 Cut Pure 300W

Instructions
Connect the Monopolar 1 output to the ESU-2400 Active port. Connect the REM port to the Dispersive 1 and Dispersive 2 ports. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



Step Number 29- 32 Monopolar 1 Output Power

Instructions
No action from user is required.


Step Number 33

Monopolar 2 Cut Pure 300W

Instructions

Move the yellow cable from the Monopolar 1 output to the Monopolar 2 output. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

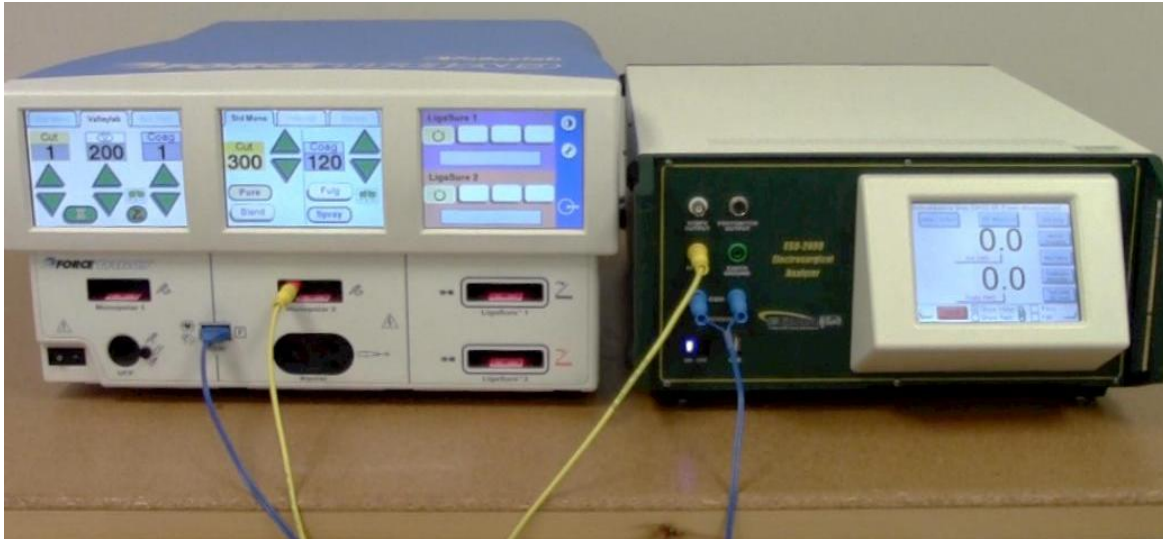

Step Number 34- 35

Monopolar 2 Output Power

Instructions

No action from user is required.

Instructions


Move the yellow cable from the Monopolar 2 output to the UFP output using the UFP Adapter. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



Instructions

No action from user is required.

| | |
|---|---------------------------------------|
| Step Number 48 | Autobipolar |
| <p>Instructions Disconnect all test cables. Connect the Autobipolar cable from the bipolar output to the ESU-2400 Active and Dispersive 1 ports. Connect a bipolar footswitch to the ForceTriad™. Follow the ESU-2400 on-screen instructions. Select the Pass or Fail checkmark and then the right arrow to advance to the next step.</p> | |
| <p>Connections</p>  | |
| Step Number 49 | Autobipolar Footswitch mode. |
| <p>Instructions Activate the ForceTriad™ using a bipolar footswitch. Then release the footswitch. The generator output should stay on. Disconnect one test cable from the ESU-2400, the bipolar output should turn off. If so, press the Pass Check mark on the ESU-2400 and the test will automatically proceed.</p> | |
| Step Number 50-64 | Autobipolar and Manual Bipolar tests. |
| <p>Instructions No action from user is required.</p> | |


Step Number 65

LigaSure™ 1 1.0A

Instructions

Disconnect bipolar test cable. Connect the LigaSure™ 1 output to the ESU-2400 Active and Dispersive 1 ports . On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

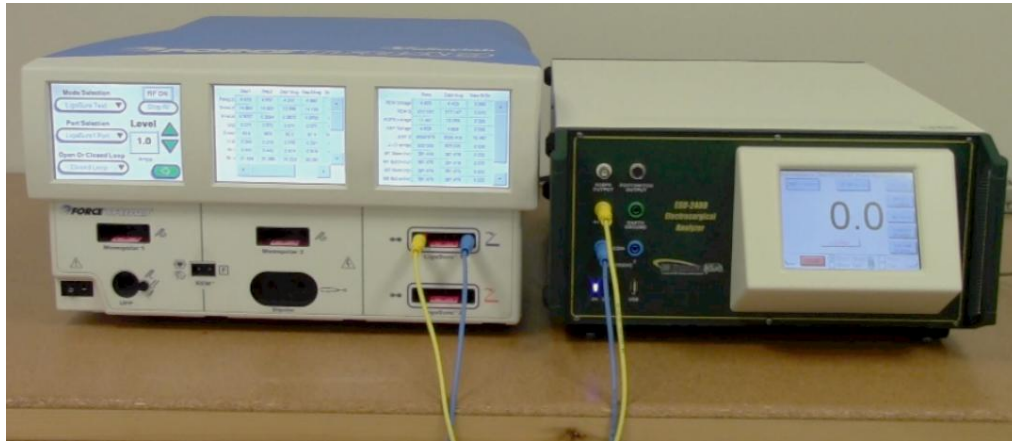

Step Number 66-67

LigaSure™ 1 Output test

Instructions

No action from user is required.

Step Number 68

LigaSure™ 2 1.0A

Instructions

Move the LigaSure™ 1 output cables to the LigaSure™ 2 output. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



Step Number 69-70

LigaSure™ 2 Output test

Instructions

No action from user is required.

Step Number 71

LigaSure™ 2 Left Tine Leakage

Instructions

Remove the blue cable from the right tine LigaSure™ 2 output and ESU-2400 Dispersive 1 port. Connect the ESU-2400 Earth Ground port to the Ground lug on the back of the ForceTriad™. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections




Step Number 72

LigaSure™ 2 Right Tine Leakage

Instructions

Move the test cable from the LigaSure™ 2 Left tine to the right tine. Do not change any connections on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections

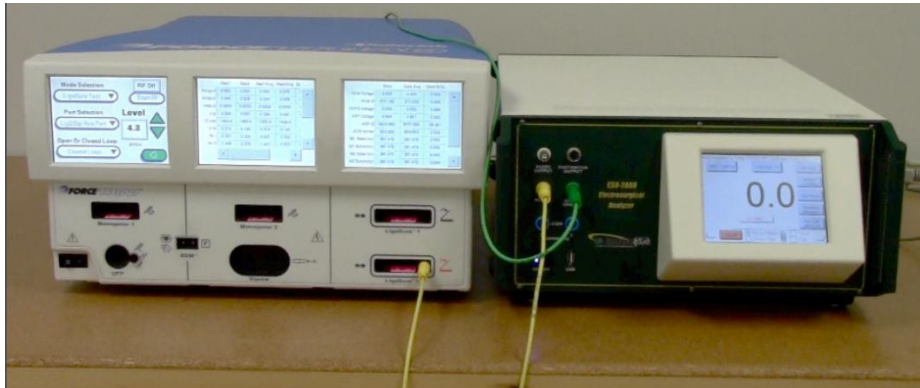

Step Number 73

LigaSure™ 1 Left Tine Leakage

Instructions

Move the test lead from the LigaSure™ 2 output to the LigaSure™ 1 Left tine. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections




Step Number 74



LigaSure™ 1 Right Tine Leakage


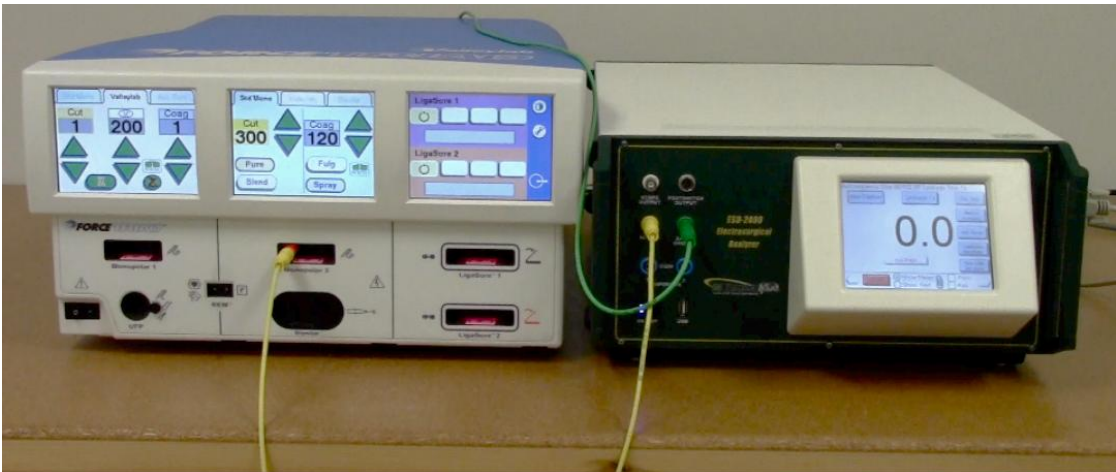
Instructions

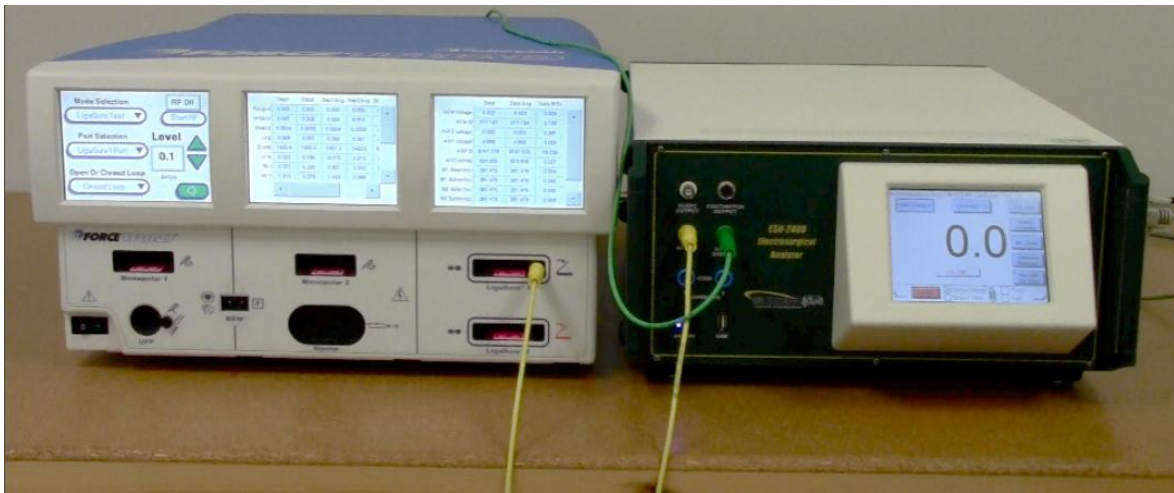

Move the test cable from the LigaSure™ 1 Left tine to the right tine. Do not change any connections on the ESU-2400. On the ESU-2400 press the Show Meter button and then the Capture button.

Connections



| | |
|---|----------------------------|
| Step Number 75 | Bipolar Left Tine Leakage |
| <p>Instructions Move the test lead from the LigaSure™ 1 output to the Bipolar Left Tine output. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 76-77 | Bipolar Left Tine Leakage |
| <p>Instructions No action from user is required.</p> | |
| Step Number 78 | Bipolar Right Tine Leakage |
| <p>Instructions Move the test lead from the Bipolar Left Tine output to the Bipolar Right Tine output. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 79-80 | Bipolar Right Tine Leakage |
| <p>Instructions No action from user is required.</p> | |

| | |
|--|---------------------|
| Step Number 81 | Monopolar 1 Leakage |
| <p>Instructions Move the test lead from the Bipolar Right Tine output to the Monopolar 1 output. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 82-85 | Monopolar 1 Leakage |
| <p>Instructions No action from user is required.</p> | |
| Step Number 86 | Monopolar 2 Leakage |
| <p>Instructions Move the test lead from the Monopolar 1 output to the Monopolar 2 output. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 87-90 | Monopolar 2 Leakage |
| <p>Instructions No action from user is required.</p> | |


| | |
|---|------------------------|
| Step Number 91 | UFP Leakage |
| <p>Instructions Move the test lead from the Monopolar 2 output to the UFP output using the UFP Adapter. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 92-95 | UFP Leakage |
| <p>Instructions No action from user is required.</p> | |
| Step Number 96 | Patient Return Leakage |
| <p>Instructions Remove the test lead from the UFP output. Connect the REM port to the ESU-2400 Dispersive 1 and Dispersive 2 ports. On the ESU-2400 press the Show Meter button and then the Capture button.</p> | |
| <p>Connections</p>  | |
| Step Number 97-100 | Patient Return Leakage |
| <p>Instructions No action from user is required.</p> | |

Step Number 101

Low Frequency Leakage

Instructions

Disconnect the test leads from the ESU-2400 and ForceTriad™. Using a safety analyzer, perform the low frequency leakage measurements as described on the ESU-2400 screen. Check the appropriate Pass or Fail checkbox based on the results of the low frequency leakage tests. Then press the Right Arrow to continue the test.

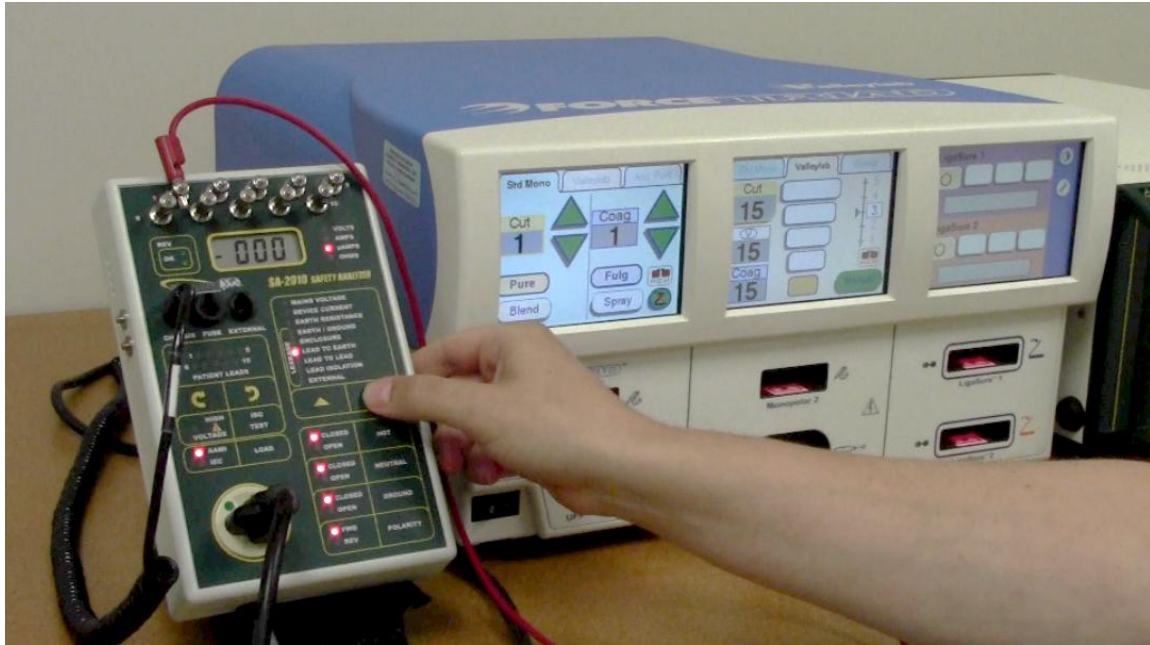

Step Number 102

Valleylab Exchange

Instructions

Dock the ForceTriad™ to Valleylab Exchange to log any changes to the system. The ForceTriad™ Energy Platform User's Guide contains instructions for docking to Valleylab Exchange. Check the appropriate Pass or Fail checkbox based on the status of the Valleylab Exchange and then press the Right Arrow to show the test results.

The test can then be printed or saved.

Note: PDF files cannot be viewed on the ESU-2400.

.tst files can only be viewed on the ESU-2400.

USER GUIDE REVISIONS

| Revision # | Revisions Made |
|-------------------|---|
| Rev 01 | Origination |
| Rev 02 | Pictures Updated, Format Updated, Miscellaneous Edits |

NOTES

NOTES



**BC GROUP INTERNATIONAL, INC.
3081 ELM POINT INDUSTRIAL DRIVE
ST. CHARLES, MO 63301
USA**

**1-800-242-8428
1-314-638-3800**

**www.bcgrouptl.com
sales@bcgrouptl.com**

**ESU-2400 User Guide
ForceTriad™ PM Procedure Using ESU-2400 Autosequence
07/12 – Rev 02**

**Copyright © 2012
Made in the USA**